

Light Rolltalk



Light Rolltalk with text-based setup



Light Rolltalk, adapted for symbol users

Overview

Light Rolltalk is a communication aid for people with speech disabilities.

Light Rolltalk also lets you operate a wheelchair, control your environment, and use mobile phones, e-mail and the Internet.

The system comes complete with a range of pre-defined settings that you can then adapt to suit your personal needs.

The functions and content of Light Rolltalk are defined on the basis of the user's situation and interests.

Communication via Rolltalk

Quick communication – Selecting fields with pre-defined images, text or synthetic speech makes communication quick and easy. You can use the images supplied with Rolltalk, photographs, PCS, Bliss symbols or Pictograms.

Word processing – The system comes complete with a selection of screen keyboards with suggested words and letters. Together with the synthetic speech function, which reads text aloud, this provides unlimited communication options.

It is also possible to combine the quick communication and word processing functions.

Rolltalk Workshop

Light Rolltalk contains Rolltalk Workshop, which is an intuitive and flexible content program. Rolltalk Workshop provides a unique opportunity to customise your personal aid. Rolltalk is supplied with pre-defined user programs that make it easy to start using the system, and solutions that simplify the customisation of Rolltalk to suit the wishes and needs of the individual user. The Rolltalk Workshop ensures that Light Rolltalk can develop and grow with the user's ability.

Light Rolltalk basic package

000170

Supplementary equipment

Environment control	000140	Head mouse, Tracker Pro	000730
Rolltalk control module	000120	Switches	misc.
Light Rolltalk table stand	000173	Wireless switchbox	001310/1
Daessy wheelchair bracket	000174	Bliss panel with Bliss archive	000
Transformer el. wheelchair mount	000381	PCS with ASKeladden	000281
Battery manual wheelchair	000175	Pictogram archive	000230
Bluetooth CF card	000755		

The Light Rolltalk package contains:

Programsnekker software
Windows XP, Wireless LAN (Internet)
Switch module and special loudspeakers
Charger, internal battery
Microphone, mouse, keyboard
Daessy fitting arrangements
User guides

Light Rolltalk specifications

Dimensions (cm): 22 x 14 x 2,5
Weight: 0.9 kg
Operating system: Windows XP
RAM: 512 MB
Processor: Intel celeron 900 M Hz
Hard disk: 40 GB
Screen: 7" touch screen

Speech synthesis

Users can choose between several gender-specific speech synthesis options.

Mobile phone and SMS

SMS messages can be written from the screen keyboard or by using symbols. SMS messages are sent as text, while SMS messages received can be viewed on screen or read aloud.

Internet and e-mail

E-mails can be written from the screen keyboard or by using symbols. Users can also choose to translate incoming e-mails into symbols or have them read aloud.

Environment control

The environment control feature of Rolltalk provides users with a unique method of controlling their environments. All doors, windows, etc. can be fitted with motors so that they can be operated from Rolltalk. In addition, all appliances with IR remote control – such as TVs, entertainment electronics, etc. – can be controlled via the intelligent IT environment control feature in Rolltalk.

Operate wheelchair

With the Rolltalk control module and Light Rolltalk, you can operate most electric wheelchairs and adjust the seat position.

Operating Rolltalk

Light Rolltalk can be operated via a touchscreen or head mouse, a joystick or switches. A conscious movement is sufficient to operate Light Rolltalk.

Support / Remote assistance

All Light Rolltalk units are prepared for remote assistance via the Internet and this is simple to set up before using the product for the first time. It means that all users who are linked to the Internet can receive support directly to their machines from our customer centres.



PCS-archive with AACe communication program



Table stand



Head mouse, Tracker Pro



Div. Switches



External battery pack for manual wheelchair

Falck Igel AS

Risør
Buvikveien 22, 4950 Risør
P.b. 123, NO-4952 Risør
Tel. +47 37 14 94 50
Fax: +47 37 14 94 70

post@falckigel.no
www.falckigel.no

Distributor Ireland & Northern Ireland

CRC Medical,
Unit 19, Seatown Business Campus
Seatown Road, Swords
Co Dublin
Tel. +353 1 883 3293
www.crcmedical.com